



Uni-Gold™ Recombigen® HIV Control

HIV Positive and Negative Controls

Catalogue No: 1206530

NAME AND INTENDED USE

Uni-Gold™ Recombigen® HIV Controls are intended for use **only** with the Uni-Gold™ Recombigen® HIV test (Catalogue No. 1206503).

SUMMARY

Controls should be used in conjunction with Good Laboratory Procedures. Controls should be run at the beginning of a shift, with a change of operator or change of Uni-Gold™ Recombigen® HIV test lot.

PRINCIPLES OF THE PROCEDURE

Uni-Gold™ Recombigen® HIV Controls have been designed for use with the Uni -Gold™ Recombigen® HIV assay to validate the correct performance of the device in the hands of the user.

Uni-Gold™ Recombigen® HIV Positive Control is prepared from inactivated human serum or plasma. It is negative for HbsAg and anti-HCV by U.S. FDA licensed test procedures. Source materials are reactive for antigens to HIV-1.

Positive Controls do not have assigned quantitative values, each lot of material has been designed to produce a positive reaction within a target range, when tested on the Uni-Gold™ Recombigen® HIV assay.

Uni-Gold™ Recombigen® HIV Negative Controls are prepared from defibrinated delipidised human serum which has been screened for Anti-HIV-1 and HIV-2, HbsAg and Anti-HCV. Uni-Gold™ Recombigen® HIV Negative Controls have been designed to give a negative reaction when tested on the Uni -Gold™ Recombigen® HIV assay.

REAGENTS

- Uni-Gold™ Recombigen® Positive Control: 1 vial (500 µl) with red cap.
- Uni-Gold™ Recombigen® Negative Control: 1 vial (500 µl) with black cap.

Positive control contains human serum or plasma reactive for antibody for HIV-1. Source material has been treated with beta-propiolactone and ultraviolet irradiation (BP/UV). Negative control contains defibrinated delipidised human serum. Both the positive and negative controls contain 0.1% Sodium Azide as a preservative.

WARNINGS AND PRECAUTIONS

FOR *IN VITRO* DIAGNOSTIC USE ONLY

CAUTION: Handle Uni-Gold™ Recombigen® HIV Controls and all human blood products as though capable of transmitting infectious agents.

Safety Precautions

1. Do not pipette by mouth.
2. Do not eat, drink, apply cosmetics or handle contact lenses where specimens are being tested.
3. Clean any spillages by immediately and thoroughly wiping up with a suitable disinfectant such as 1% sodium hypochlorite solution.
4. Handle carefully and dispose of all specimens, controls and materials as though they contained infectious agents.

Handling Procedures

1. Do not use Uni-Gold™ Recombigen® HIV Controls beyond the expiration date.
2. Avoid microbial contamination of the controls when opening and closing the vials.

STORAGE INSTRUCTIONS

1. Store Uni-Gold™ Recombigen® HIV Controls at 2-8°C / 35.6–46.4oF
2. Store in the upright position at all times to prevent leakage.
3. Ensure cap is securely fastened when controls are not in use.
4. Once opened Uni -Gold™ Recombigen® HIV Controls are stable for one month.
5. Record the date to discard the controls (one month after opening) on the space provided on the box. This date cannot be post the expiry date of the controls printed on the box.

INDICATIONS OF REAGENT

INSTABILITY OR DETERIORATION

Alterations in physical appearance may indicate instability or deterioration of Uni-Gold™ Recombigen® HIV Controls. Solutions that are visibly turbid should be discarded in accordance with safety procedures.

PROCEDURE

Materials Required but not Provided

Uni-Gold™ Recombigen® HIV pack insert. (Pt No. 545-030)

Instructions for use


1. Read the Uni-Gold™ Recombigen® HIV package insert prior to using Uni -Gold™ Recombigen® HIV Controls.

2. Remove from storage at 2-8°C and allow the controls to reach room temperature prior to use. (Return controls to storage at 2-8°C after use).
3. Mix contents of vials by gentle swirling or inversion.
4. Refer to Test Procedure section of the Uni-Gold™ Recombigen® HIV pack insert.
5. Treat Uni-Gold™ Recombigen® HIV positive and negative controls as “patient specimens”.

Quality Control


Results should be determined in the same manner as that used for unknown specimens when testing using the Uni-Gold™ Recombigen® HIV assay.

INTERPRETATION OF RESULTS

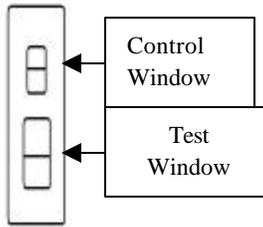


(+) POSITIVE RESULT

Two lines appear
A line in both test and control window indicates a positive result

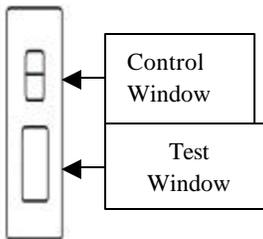

(-) NEGATIVE RESULT

Only one line appears in the control window
A line in the control window only indicates a negative test result

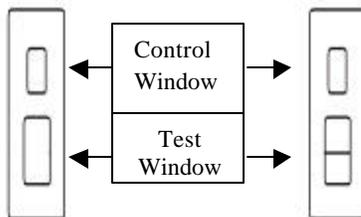

INVALID RESULT

No Line appears in the Control window
Irrespective of a line developing in the test window. This is an invalid result. **Repeat Test with a fresh device and a new patient specimen.**

LIMITATIONS OF THE PROCEDURE

Uni-Gold™ Recombigen® HIV Controls are only validated for use with Uni-Gold™ Recombigen® HIV assay.

1. TEST PROCEDURES and INTERPRETATION OF TEST RESULTS section in the Uni-Gold™ Recombigen® HIV assay pack insert must be adhered to when testing Uni-Gold™ Recombigen® HIV Controls.
2. Deviations from the procedure outlined in the Uni-Gold™ Recombigen® HIV test pack insert may produce unreliable results.
3. Uni-Gold™ Recombigen® HIV Controls are intended for use in undiluted form.
4. Adverse shipping and storage conditions or use of expired reagents may produce erroneous results.

EXPECTED RESULTS

Uni-Gold™ Recombigen® HIV Controls do not have assigned values. Results should be determined in the same manner as used for unknown specimens when testing with the Uni-Gold™ Recombigen® HIV test. Each laboratory should determine its own range of acceptable values.

SPECIFIC PERFORMANCE CHARACTERISTICS

Uni-Gold™ Recombigen® HIV Controls have been validated for use with the Uni-Gold™ Recombigen® HIV test.

A positive reaction is produced when positive controls are run in the same manner as unknown specimens. Conversely a negative reaction is produced when negative controls are run in the same manner as unknown specimens.

All testing must be carried out in accordance with the Uni-Gold™ Recombigen® HIV assay pack insert.

GLP must be followed when using Uni-Gold™ Recombigen® HIV Controls.

Manufacturer:
Trinity Biotech plc
IDA Business Park,
Bray,
Co. Wicklow,
Ireland.

Tel: 353-1-276 9800

Fax: 353-1-2769888

E-mail: info@trinitybiotech.ie

Web: www.trinitybiotech.com

Distributor;
Trinity Biotech USA
2823 Girts Road,
Jamestown,
NY 14702-1059

Tel: (716) 483 3851

Fax: (716) 488 1990

