

## A PATIENT'S GUIDE TO TREATMENT WITH RADIESSE™

The logo for Radiesse, featuring the word "Radiesse" in a blue serif font with a trademark symbol. A blue wavy line underlines the text.

Read all the information before you are treated with Radiesse.

- Keep this information. You may need to read it again.
- If you have any questions, please ask your doctor.

BioForm Medical, Inc.  
1875 South Grant Street  
Suite 110  
San Mateo, CA 94402

May 2006

## **Introduction**

This information will help you decide whether treatment with Radiesse is right for you. This information does not take the place of a discussion with your doctor but will answer some questions about treatment with Radiesse.

Please read this information and discuss any questions with your doctor. Only you and your doctor can decide whether Radiesse is right for you.

## **Glossary**

<i>Anesthetic:</i>	A substance that causes loss of feeling or awareness. A topical or local anesthetic causes temporary loss of feeling in a part of the body, and may make your treatment more comfortable.
<i>Calcium Hydroxylapatite</i>	Calcium Hydroxylapatite is a biocompatible (compatible with living systems), biodegradable (dissolves in the body) material. Calcium hydroxylapatite is identical in composition to the mineral portion of teeth and bone.
<i>Side effect:</i>	An undesirable event caused by use of the product.

## Background Information

### What is Radiesse?

Radiesse is a longer-lasting injectable skin builder that does more than just temporarily fill facial lines and wrinkles. Using advanced Microsphere Technology™, Radiesse helps rebuild new skin below the surface to restore the fullness and youthful shape of your face without surgery. Radiesse doesn't just fill lines and wrinkles. It rebuilds them to let you stay looking younger longer. Radiesse has an extensive history of safe use in otology (ears), laryngeal (vocal chords), and dental and orthopedic applications.

### What is Radiesse used for?

Radiesse is used to fill and help rebuild facial lines and wrinkles including nasolabial folds—the creases that extend from the corner of your nose to the corner of your mouth. It can also be used to reshape your nose, cheeks, chin, brow, or jaw line. Radiesse will not correct the underlying causes of facial wrinkles.

### Can Radiesse work for me?

Talk to your doctor about your medical history when deciding on treatment options. You should not use Radiesse if you are allergic to any ingredient of Radiesse. You should also not choose Radiesse if you desire only short-term results.

### How does Radiesse work?

Once injected, Radiesse immediately adds fullness to your face giving you a visible result at the first treatment session. Then tiny man-made Microspheres form a scaffold, around which new skin tissue begins to build. These unique Microspheres are made of a natural material called calcium hydroxylapatite in a water based gel carrier (see glossary).

### How long do treatment effects last?

Although treatment effects will differ for each person, in a clinical study, the treatment lasted for at least one year for all patients.

### Do injections of Radiesse hurt?

As with any injection, injections with Radiesse may hurt. Radiesse is injected in small amounts using a very fine needle. Your doctor may apply a topical or local anesthetic.

### Are skin tests needed before treatment with Radiesse?

No skin testing is required prior to use.

### Should I tell my healthcare provider what medications I am taking?

Yes. You should tell your doctor about all the medicines you are taking, even over the counter medicines or treatments. If you are taking blood thinners or medications that may interfere with clotting of the blood, such as aspirin, you might be more likely to have bruising or bleeding at the injection site. There have been no studies of possible interactions between Radiesse and drugs or other substances or implants.

### What can I expect to happen at a treatment session?

- Your doctor will answer all of your questions and prepare you for the treatment.
- The area where the injections will be given will be cleaned with an antiseptic.
- You and your doctor will determine if an anesthetic is needed.

- Radiesse will be injected in small amounts into the skin using a very fine needle.
- An ice pack may be applied to the treatment area before or after treatment to help reduce swelling.

### **What are the possible side effects of treatment with Radiesse?**

Talk to your doctor about the possible side effects of Radiesse. The most common side effects are redness, bruising, or swelling. These side effects generally last for a short time and are mild in nature. As with all procedures that involve an injection through the skin, there is a risk of infection. However, no infections have been reported in the clinical study of Radiesse. Report any side effects you may experience to your doctor.

### **What can I expect after treatment?**

Immediately following a treatment session with Radiesse redness, bruising or swelling may occur in the treatment area. These signs usually go away in a short period and are generally mild in nature. An ice pack may be applied to the treatment area to help reduce swelling. Your health care provider will give you specific post-treatment care instructions.

For approximately 24 hours after treatment:

- Avoid significant movement or massage of the treated area.
- Do not apply makeup.
- Avoid extensive sun or heat exposure

After you leave the office, you may experience redness, bruising or swelling for a few days. Not all patients experience these, but please note that such effects can occur and are not out of the ordinary for such a treatment.

You may be able to feel the area where Radiesse was injected for some period after injection. Over time, the injected area will feel more and more like your own tissue.

### **How quickly can I get back to my daily activities?**

Most patients feel comfortable going back to their normal activities immediately after treatment with Radiesse.

### **How many treatments are required?**

Your doctor will decide with you the number of treatment sessions and the amount of Radiesse you will need at each treatment session. Generally, significant changes may be achieved in a first treatment. A touch-up may be required to achieve optimal results.

### **Without touch-up injections, how will my skin look?**

Your skin will begin to gradually look like it did before treatment as you near the end of the time period Radiesse usually lasts. Periodic touch up injections will help you maintain the look you have after treatment.

### **What other things do I need to know?**

The Microspheres in Radiesse can be seen in X-rays and CT Scans. It is very important that you inform your doctor and other health care professionals that you have Radiesse injected in your face. Even though Radiesse can be seen in X-rays and CT Scans, there is not a high risk that it would cause your doctor concern, as long as s/he knows that you have had Radiesse injected in your face.

BioForm Medical Inc.  
4133 Courtney Road, Suite #10  
Franksville, WI 53126 U.S.A.  
Telephone: 262-835-9800  
Fax: 262-835-9311  
E-Mail: [info@bioformmedical.com](mailto:info@bioformmedical.com)