

# Capillary Blood Glucose Testing in Hospital Settings

Meeting of the Clinical Chemistry and Clinical Toxicology Devices Panel  
March 30, 2018

Courtney H. Lias, Ph.D.

Director, Division of Chemistry and Toxicology Devices  
Office of In Vitro Diagnostics and Radiological Health  
Center for Devices and Radiological Health  
Food and Drug Administration




# Hospital Glucose Meters

Blood glucose monitoring systems (BGMS) have become critical tools in hospital settings

- easy to use
- accessible for bedside testing
- fast results
- small blood volume required
- less expensive
- portable, etc.

# Hospital Glucose Meters

- Originally home use devices
- Migrated into healthcare settings

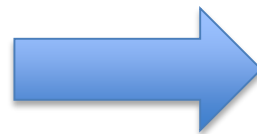

# Hospital Glucose Meters


- Increased stakeholder discussion of BGMS accuracy requirements for different uses
- Different patient populations have different needs
  - Routine glucose monitoring (diabetic and non-diabetic)
  - Glucose testing to inform insulin dosing

# Glycemic Control Protocols

- Van den Berghe group (Leuven, Belgium) demonstrated that reducing hyperglycemia in intensive care patients led to better clinical outcomes
- Demonstrated lower mortality in intensive care patients when glucose levels were managed to a strict range of 80-110 mg/dL using infused insulin by expert nursing staff.
- Practice known as “tight glycemic control.”



# Glycemic Control Protocols

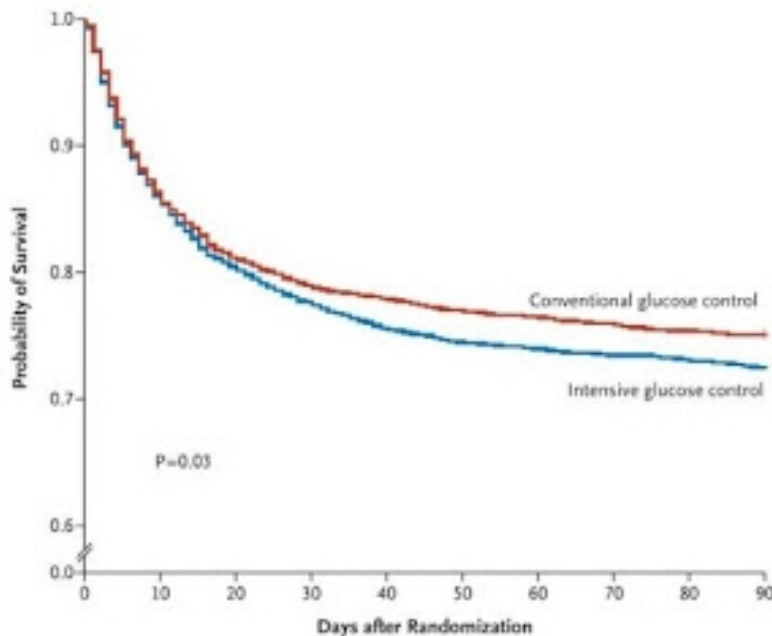

- Leuven results not replicated in some other large studies
- NICE-SUGAR\* discontinued after an increase in mortality due to hypoglycemia was observed in the tight glycemic control arm

\* Normoglycemia in Intensive Care Evaluation and Survival Using Glucose Algorithm Regulation

The NICE-SUGAR Study Investigators. Intensive versus Conventional Glucose Control in Critically Ill Patients. *N Engl J Med.* 2009; 360:1283-1297

Van den Berghe, G. et. al. Intensive Insulin Therapy in Critically Ill Patients: NICE-SUGAR or Leuven Blood Glucose Target. *J Clin Endocrinol Metab.* 2009; 94(9): 3163-3170

# Glycemic Control Protocols



## Why did NICE-SUGAR fail?

- varying levels of insulin dosing expertise in the study staff
- different target ranges for blood glucose by site
- different nutritional strategies
- different types of insulin administration
- different specimen types (e.g., venous/arterial vs. capillary)
- different instruments used to measure blood glucose.

Leuven:  
Bedside Blood Gas  
Analyzers

NICE-SUGAR:  
Variable, many sites  
used capillary BGMS

## POCT<sub>12</sub>-A<sub>3</sub>

Point-of-Care Blood Glucose Testing in  
Acute and Chronic Care Facilities; Approved  
Guideline—Third Edition


### Performance goal:

1. 95% of the results must have differences from the laboratory analyzer less than 12 mg/dl below 100 mg/dl and less than 12.5% above 100 mg/dl, and
2. The sum of the number of individual results with errors that exceed 15 mg/dl below 75 mg/dl and exceed 20% at glucose concentrations at or above 75 mg/dl should not exceed 2% of all results.



# FDA BGMS Guidance

- 2010: FDA public meeting on glucose meter accuracy
- 2014: FDA draft BGMS Guidance
- 2016: FDA final BGMS Guidance

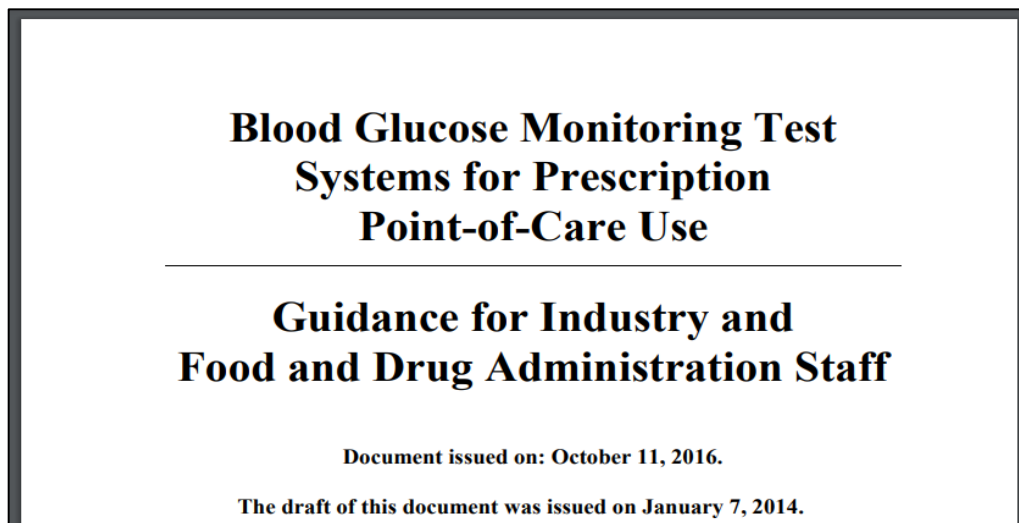

# FDA BGMS Guidance


## BGMS studies:

- Labeling specifies claimed healthcare settings and patient populations claimed
  - Capillary vs. venous blood
  - Physician’s office, hospital, ambulance, etc.
- Manufacturer validates the device in this setting and population



# BGMS use in Intensive Care Settings

- Home use meters validated in a healthier population
- Limitations against using the devices in certain populations, including patients receiving intensive medical intervention/therapy

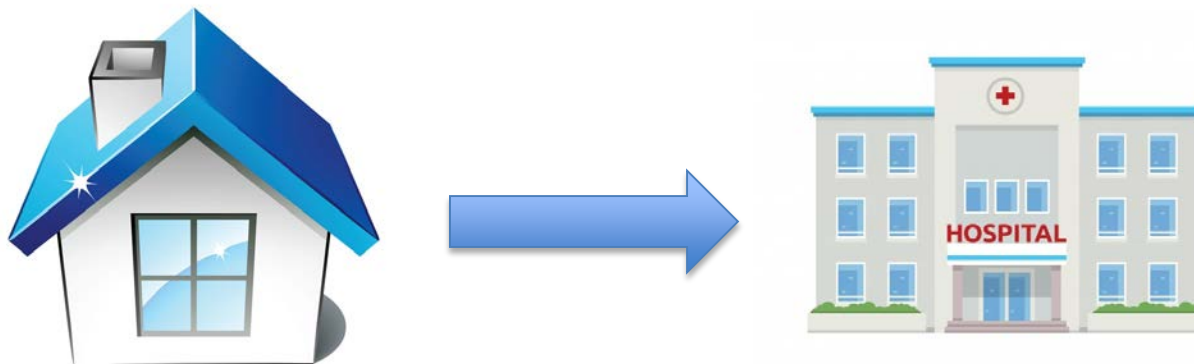

# BGMS use in Intensive Care Settings



- CMS regulates laboratories and laboratory testing under the Clinical Laboratory Improvement Act (CLIA)
- Off label use = high complexity
- Increased regulation
- Demanding personnel training requirements



# StatStrip Clearance

- FDA encouraged manufacturers to seek this claim
- 2014: Nova Biomedical StatStrip BGMS FDA clearance and CLIA waiver
  - Venous, arterial, neonatal heel stick in all hospitalized patients
  - Capillary blood still has limitation in patients receiving intensive care



# CLIA Waiver for BGMS

- We recognize the burden of bedside glucose testing in hospitals when CLIA waived BGMS devices are not available
- We also understand that being able to make capillary blood measurements in all hospitalized patients using FDA cleared and CLIA waived BGMS would be more convenient and feasible for hospital staff.





- Present capillary BGMS data in intensively treated patients
- Provide transparency about the performance of these devices for this use
- Hear from our Panel and the community on this topic

