

8.9 iCASI Mobile

iCASI Mobile pertains to the iCASI App and the iCASI Mobile Optimized Website. Both are covered in the following two sections: 8.9.1 and 8.9.2.

NOTE: The iCASI App is currently only available on iOS7 and iOS8. iOS9 and Android operating system Android Jelly Bean(API 16) and Android 5.1 Lollipop(API 22).

8.9.1 iCASI App

To provide the Blood Center with further options for the use of the iCASI functionality, the iCASI APP has been developed. Using this feature the Blood Center allows the donor to access the iCASI module from a cellular device or tablet via an application (App) prior to registration of the donor, eligibility check, and donation type selection at the collection site. With this, a 2D barcode is stored within the app for the donor containing basic identifying information to be used for matching the donor record during Registration to the information provided during the iCASI session. This barcode is usable only on the calendar day on which it was created. The donor must bring this barcode with them to the collection site in order to have the iCASI responses evaluated in Health History; otherwise, the questionnaire must be repeated at the collection site.

Processing the Donor within the Donor-ID application remains the same process as with regular iCASI. Refer to section 8.6.3, in this manual, for additional information.