




COVID-19 Test Home Collection Kit

Fast, safe, and accurate results from the comfort of your home.



Nasal Swab

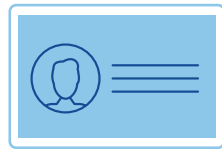

Preparation

Gather your test kit, photo ID, a pen and Telehealth visit registration code from the confirmation email. If you cannot locate your confirmation email, please call 1-844-KRO-CV19 for support.



Two-Way Video

Test the Telehealth platform on your mobile device or desktop by clicking the test link in the confirmation email.




Prior to Test

Wash hands thoroughly for 20 seconds prior to launching your Telehealth visit.



Do not open kit contents until instructed to do so during your Telehealth Visit.

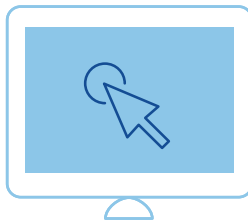

Launch Telehealth Visit

Use the unique link and registration code provided in your confirmation email to enter the waiting room for your Telehealth visit. This is required for the lab to process your test kit.



Specimen Collection

A licensed healthcare professional will join you to give directions and observe the collection of your specimen.




Label

Ensure the tube is tightly sealed and use the included barcode to label your specimen tube. Write today's date on label.

krogerhealth.com/covidtestkit

3
Package

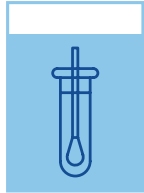

Pack Your Kit

Seal the tube in the Biohazard bag and wipe with alcohol swab provided. Place bag in the original box and seal in the UN 3373 return envelope.

Ship to Lab


Visit [ups.com/dropoff](https://www.ups.com/dropoff) to find a location closest to you. Drop off packaged test kit using the pre-paid shipping label. This must be done the same day as your visit.

Await Results

Within 72 hours, your test results will be provided to you. Positive results will be communicated via phone call with a healthcare professional.

CONTINUE TO STAY SAFE

While waiting for your results, maintain physical distancing and monitor for additional symptoms that may require immediate medical attention. Examples of such symptoms may include difficulty breathing that leaves you unable to speak full sentences, persistent pain or pressure in the chest, new confusion or inability to wake, or bluish lips or face.

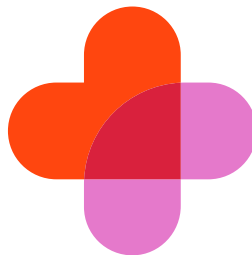

For more information about your Home Collection Kit, or how to get tested, please visit [krogerhealth.com/covidtestkit](https://www.krogerhealth.com/covidtestkit)

- This home sample collection kit has not been FDA cleared or approved.
- This home sample collection kit has been authorized by FDA under an EUA. This home sample collection kit has been authorized only for the home collection and maintenance of nasal swab specimens as an aid in detection of nucleic acid from SARS-CoV-2, not for any other viruses or pathogens.
- This home sample collection kit in combination with the authorized test is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostic tests for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Act, 21 U.S.C. § 360bbb-3(b)(1), unless the authorization is terminated or revoked sooner.

