

# FACT SHEET FOR HEALTHCARE PROVIDERS

Color Health, Inc.

Color SARS-CoV-2 RT-LAMP Diagnostic Assay

Updated: April 14, 2021

Coronavirus  
Disease 2019  
(COVID-19)

This Fact Sheet informs you of the significant known and potential risks and benefits of the emergency use of the Color SARS-CoV-2 RT-LAMP Diagnostic Assay.

The Color SARS-CoV-2 RT-LAMP Diagnostic Assay is authorized for use with certain respiratory specimens collected from individuals suspected of COVID-19 by a healthcare provider, and anterior nasal swabs collected from any individual, including from individuals without symptoms or other reasons to suspect COVID-19 infection.

**All patients whose specimens are tested with this assay will receive the Fact Sheet for Patients: Color Health Inc.- Color SARS-CoV-2 RT-LAMP Diagnostic Assay.**

## What are the symptoms of COVID-19?

Many patients with confirmed COVID-19 have developed fever and/or symptoms of acute respiratory illness (e.g., cough, dyspnea). The current information available to characterize the spectrum of clinical illness associated with COVID-19 suggests that symptoms include cough, shortness of breath or dyspnea, fever, chills, myalgias, headache, sore throat or new loss of taste or smell. Based on what is known about the virus that causes COVID-19, signs and symptoms may appear any time from 2 to 14 days after exposure to the virus, and the median incubation period is approximately 5 days. For further information on the symptoms of COVID-19 please see the link provided in “*Where can I go for updates and more information?*” section at the end of this document.

Public health officials have identified cases of COVID-19 infection throughout the world, including the United States. Please check the CDC COVID-19 webpage (see link provided in “*Where can I go for updates and more information?*” section) for the most up to date information.

## What do I need to know about COVID-19 testing?

Current information on COVID-19 for healthcare providers is available at CDC’s webpage, *Information for*

**This test is to be performed only using certain respiratory specimens collected from individuals suspected of COVID-19 by a healthcare provider, and anterior nasal swabs collected from any individual, including from individuals without symptoms or other reasons to suspect COVID-19 infection.**

*Healthcare Professionals* (see links provided in “*Where can I go for updates and more information?*” section).

- The Color SARS-CoV-2 RT-LAMP Diagnostic Assay can be used to test nasopharyngeal swabs, oropharyngeal swabs, anterior nasal swabs, mid-turbinate nasal swabs, nasopharyngeal washes/aspirates or nasal aspirates, as well as bronchoalveolar lavage specimens from individuals suspected of COVID-19 by a healthcare provider, and anterior nasal swabs collected from any individual, including from individuals without symptoms or other reasons to suspect COVID-19 infection.
- The Color SARS-CoV-2 RT-LAMP Diagnostic Assay can be used to test anterior nasal swab specimens that are collected using either the Color COVID-19 Self-Swab Collection Kit or the Color COVID-19 Self-Swab Collection Kit with Saline when used consistent with their authorizations.
- The Color SARS-CoV-2 RT-LAMP Diagnostic Assay is only authorized for use at Color Health Inc., located at 863 Mitten Road, Burlingame, CA 94010 which is certified under the Clinical Laboratory Improvement Amendments of 1988 (CLIA), 42 U.S.C. §263a, and meets requirements to perform high complexity tests.

Specimens should be collected with appropriate infection control precautions. Current guidance is available at the CDC’s website (see links provided in “*Where can I go for updates and more information?*” section).

When collecting and handling specimens from individuals suspected of being infected with COVID-19, appropriate personal protective equipment should be

**Report Adverse events**, including problems with test performance or results, to MedWatch by submitting the online FDA Form 3500 (<https://www.accessdata.fda.gov/scripts/medwatch/index.cfm?action=reporting.home>) or by calling **1-800-FDA-1088**

# FACT SHEET FOR HEALTHCARE PROVIDERS

Color Health, Inc.

Color SARS-CoV-2 RT-LAMP Diagnostic Assay

Updated: April 14, 2021

Coronavirus  
Disease 2019  
(COVID-19)

used as outlined in the CDC *Interim Laboratory Biosafety Guidelines for Handling and Processing Specimens Associated with Coronavirus Disease 2019 (COVID-19)*. For additional information, refer to CDC *Interim Guidelines for Collecting, Handling, and Testing Clinical Specimens from Persons Under Investigation (PUIs) for Coronavirus Disease 2019 (COVID-19)* (see links provided in “Where can I go for updates and more information?” section).

## What does it mean if the specimen tests positive for the virus that causes COVID-19?

A positive test result for COVID-19 indicates that RNA from SARS-CoV-2 was detected, and therefore the patient is infected with the virus and presumed to be contagious.

Test results should always be considered in the context of clinical observations and epidemiological data in making a final diagnosis and patient management decisions. Patient management should be made by a healthcare provider and follow current CDC guidelines.

The Color SARS-CoV-2 RT-LAMP Diagnostic Assay has been designed to minimize the likelihood of false positive test results. However, in the event of a false positive result, risks to patients could include the following: a recommendation for isolation of the patient, monitoring of household or other close contacts for symptoms, patient isolation that might limit contact with family or friends and may increase contact with other individuals with COVID-19, limits in the ability to work, the delayed diagnosis and treatment for the true infection causing the symptoms, unnecessary prescription of a treatment or therapy, or other unintended adverse effects.

Color Genomics, Inc. must follow the standard testing and reporting guidelines according to their appropriate public health authorities.

## What does it mean if the specimen tests negative for the virus that causes COVID-19?

A negative test result for this test means that SARS-CoV-2 RNA was not present in the specimen above the limit of detection. However, a negative result does not rule out COVID-19 and should not be used as the sole basis for treatment or patient management decisions. It

is possible to test a person too early or too late during COVID-19 infection to make an accurate diagnosis via Color SARS-CoV-2 RT-LAMP Diagnostic Assay.

In addition, asymptomatic people infected with COVID-19 may not shed enough virus to reach the limit of detection of the test, giving a false negative result. In the absence of symptoms, it is difficult to determine if asymptomatic people have been tested too late or too early. Therefore, negative results in asymptomatic individuals may include individuals who were tested too early and may become positive later, individuals who were tested too late and may have serological evidence of infection, or individuals who were never infected.

When diagnostic testing is negative, the possibility of a false negative result should be considered in the context of a patient’s recent exposures and the presence of clinical signs and symptoms consistent with COVID-19. The possibility of a false negative result should especially be considered if the patient’s recent exposures or clinical presentation indicate that COVID-19 is likely, and diagnostic tests for other causes of illness (e.g., other respiratory illness) are negative. If COVID-19 is still suspected based on exposure history together with other clinical findings, re-testing with an alternative method should be considered by healthcare providers in consultation with public health authorities. Additional testing may be helpful to ensure testing was not conducted too early.

Risks to a patient of a false negative test result include: delayed or lack of supportive treatment, lack of monitoring of infected individuals and their household or other close contacts for symptoms resulting in increased risk of spread of COVID-19 within the community, or other unintended adverse events.

The performance of this test was established based on the evaluation of a limited number of clinical specimens. The clinical performance has not been established in all circulating variants but is anticipated to be reflective of the prevalent variants in circulation at the time and location of the clinical evaluation. Performance at the time of testing may vary depending on the variants circulating, including newly emerging strains of

**Report Adverse events**, including problems with test performance or results, to MedWatch by submitting the online FDA Form 3500 (<https://www.accessdata.fda.gov/scripts/medwatch/index.cfm?action=reporting.home>) or by calling **1-800-FDA-1088**

# FACT SHEET FOR HEALTHCARE PROVIDERS

Color Health, Inc.

Color SARS-CoV-2 RT-LAMP Diagnostic Assay

Updated: April 14, 2021

Coronavirus  
Disease 2019  
(COVID-19)

SARSCoV-2 and their prevalence, which change over time.

## What is an EUA?

The United States FDA has made this test available under an emergency access mechanism called an Emergency Use Authorization (EUA). The EUA is supported by the Secretary of Health and Human Service's (HHS's) declaration that circumstances exist to justify the emergency use of in vitro diagnostics (IVDs) for the detection and/or diagnosis of the virus that causes COVID-19.

An IVD made available under an EUA has not undergone the same type of review as an FDA-approved or cleared IVD. FDA may issue an EUA when certain criteria are met, which includes that there are no adequate, approved, available alternatives, and based on the totality of scientific evidence available, it is reasonable to believe that this IVD may be effective in diagnosing COVID-19.

The EUA for this test is in effect for the duration of the COVID-19 declaration justifying emergency use of IVDs, unless terminated or revoked (after which the test may no longer be used).

## What are the approved available alternatives?

Any tests that have received full marketing status (e.g., cleared, approved), as opposed to an EUA, by FDA can be found by searching the medical device databases here: <https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/medical-device-databases>. A cleared or approved test should be used instead of a test made available under an EUA, when appropriate and available. FDA has issued EUAs for other tests that can be found at:

<https://www.fda.gov/emergency-preparedness-and-response/mcm-legal-regulatory-and-policy-framework/emergency-use-authorization>.

## Where can I go for updates and more information?

### **CDC webpages:**

**General:** <https://www.cdc.gov/COVID19>

### **Symptoms:**

<https://www.cdc.gov/coronavirus/2019-ncov/symptoms-testing/symptoms.html>

### **Healthcare Professionals:**

<https://www.cdc.gov/coronavirus/2019-nCoV/guidance-hcp.html>

### **Information for Laboratories:**

<https://www.cdc.gov/coronavirus/2019-nCoV/guidance-laboratories.html>

**Laboratory Biosafety:** <https://www.cdc.gov/coronavirus/2019-nCoV/lab-biosafety-guidelines.html>

### **Isolation Precautions in Healthcare Settings:**

<https://www.cdc.gov/coronavirus/2019-ncov/infection-control/control-recommendations.html>

**Specimen Collection:** <https://www.cdc.gov/coronavirus/2019-nCoV/guidelines-clinical-specimens.html>

**Infection Control:** <https://www.cdc.gov/coronavirus/2019-ncov/infection-control/index.html>

### **FDA webpages:**

**General:** [www.fda.gov/novelcoronavirus](http://www.fda.gov/novelcoronavirus)

**EUAs:**(includes links to patient fact sheet and manufacturer's instructions) <https://www.fda.gov/medical-devices/coronavirus-disease-2019-covid-19-emergency-use-authorizations-medical-devices/vitro-diagnostics-euas>

### **Color Health, Inc.**

831 Mitten Road, Suite 100  
Burlingame, CA 94010

### **Customer Support:**

[support.color.com](http://support.color.com)  
[Support@color.com](mailto:Support@color.com)  
844-352-6567

**Report Adverse events**, including problems with test performance or results, to MedWatch by submitting the online FDA Form 3500 (<https://www.accessdata.fda.gov/scripts/medwatch/index.cfm?action=reporting.home>) or by calling **1-800-FDA-1088**