

1

Activate your kit online at color.com/covid/activate.


You need your unique barcode, which you'll find on your collection tube.



Warning: Your sample will not be tested if the kit is not activated.

2


Before collecting your sample, wash your hands thoroughly for 20 seconds and dry them well.

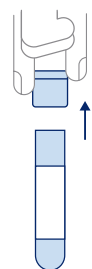

3

Remove the cap of the collection tube.

Keep the cap nearby.

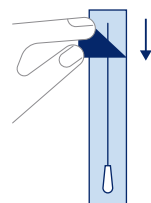


4

Open the swab package.

Starting from the handle end of the swab package, peel open the paper backing. Pull swab out of its packaging by the handle.

Note: Do not touch the soft tip with your hands or lay it down on any surface.

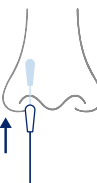

5

Rotate swab tip in first nostril, 3 times.

Insert it into one nostril just until the soft tip is no longer visible, about 1 inch. Rotate it in a circle around the inside edge of your nostril at least 3 times.

Note: Do not force the swab any further.



6

Rotate swab tip in other nostril, 3 times.

Repeat the previous step in the second nostril, using the same end of the swab.

Note: You will use the same swab for both nostrils.



7

Put swab into the collection tube.

The soft tip of the swab that went into your nose should go into the tube first.



8

Push the cap of the collection tube closed.

You're almost done! Make sure the top is closed tightly.

Note: After closing the collection tube, wash your hands thoroughly for 20 seconds and dry them well.



9

Put collection tube into the biohazard bag.

Close the bag with the adhesive seal.



10

Put biohazard bag into the return shipping mailer.

Remove the adhesive strip and seal the mailer closed. This packaging is specially designed for carrying samples suspected of carrying infectious substances. Do NOT replace it.

Note: Your return shipping has already been paid.

11

Return kit at any FedEx drop box.

Find the closest location and latest pickup times at color.com/fedex-dropbox. Your sample must be dropped off on the same day you collect it. Do NOT take the sample to a FedEx office.

Note: If you miss today's last pickup time, contact support@color.com to get a new kit.

Questions?

Visit support.color.com or contact us at support@color.com or (844) 352-6567.

Before you get started:

- Your sample must be dropped off at a FedEx drop box on the same day you collect it.
- **Do it on a weekday.** FedEx won't accept samples on a Saturday or Sunday.
- **Don't miss the last pickup.** Find the latest pickup times at color.com/fedex-dropbox.
- Make sure you wash your hands!

Your kit includes:



Swab



Collection tube



Barcode card



Biohazard bag



Return shipping mailer

Checklist before returning your kit:

- Did you activate your kit?
- Did you collect a sample from both nostrils?

Visit color.com/sample-return to learn more.

For Prescription Use Only, For In Vitro Diagnostic Use Only, For Emergency Use Authorization Only. This self-sample collection kit has not been FDA cleared or approved. This self-sample collection kit has been authorized by FDA under an EUA for use by designated laboratories.

This self-sample collection kit has been authorized only for the self-collection and maintenance of nasal specimens as an aid in detection of nucleic acid from SARS-CoV-2, not for any other viruses or pathogens.

This self-sample collection kit is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Federal Food, Drug, and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the authorization is terminated or revoked sooner.

color

COVID-19 Self-Swab Collection Kit

At-home Instructions

color

For In Vitro Diagnostic Use, For Prescription Use Only, For Use Under Emergency Use Authorization Only

Your kit includes:




Swab



Collection tube



Barcode card

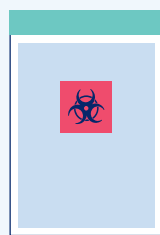

Biohazard bag

1

Activate your kit online at color.com/covid/activate.

You need your unique barcode, which you'll find on your collection tube.



Warning: Your sample will not be tested if the kit is not activated.


2

Before collecting your sample, wash your hands thoroughly for 20 seconds and dry them well.

3

Remove the cap of the collection tube.

Keep the cap nearby.

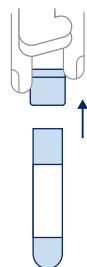


4

Open the swab package.

Starting from the handle end of the swab package, peel open the paper backing. Pull swab out of its packaging by the handle.

Note: Do not touch the soft tip with your hands or lay it down on any surface.

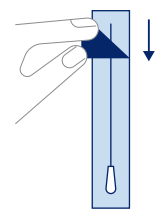


5

Rotate swab tip in first nostril, 3 times.

Insert it into one nostril just until the soft tip is no longer visible, about 1 inch. Rotate it in a circle around the inside edge of your nostril at least 3 times.

Note: Do not force the swab any further.

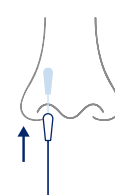

6

Rotate swab tip in other nostril, 3 times.

Repeat the previous step in the second nostril, using the same end of the swab.

Note: You will use the same swab for both nostrils.



7

Put swab into the collection tube.

The soft tip of the swab that went into your nose should go into the tube first.



8

Push the cap of the collection tube closed.

You're almost done! Make sure the top is closed tightly.

Note: After closing the collection tube, wash your hands thoroughly for 20 seconds and dry them well.



9

Put collection tube into the biohazard bag.

Seal the bag closed.



10

Bring the biohazard bag to your designated test collection point.

Your COVID-19 test collector will show you where to deposit your sample.

Note: If you miss today's last pickup time, ask your COVID-19 test collector for a new kit.

Before you get started:

- Your sample must be dropped off at your designated test collection point on the same day you collect it.
- **Make sure there is a collection today.**
Check with your COVID-19 test collector that samples will be accepted today.
- **Don't miss the last pickup.**
Check with your COVID-19 test collector on the latest collection time.
- **Make sure you wash your hands!**

Checklist before returning your kit:

- Did you activate your kit?
- Did you collect a sample from both nostrils?

Visit color.com/sample-return to learn more.

Questions?

Visit support.color.com or contact us at support@color.com or (844) 352-6567.

For Prescription Use Only, For In Vitro Diagnostic Use Only, For Emergency Use Authorization Only. This self-sample collection kit has not been FDA cleared or approved. This self-sample collection kit has been authorized by FDA under an EUA for use by designated laboratories.

This self-sample collection kit has been authorized only for the self-collection and maintenance of nasal specimens as an aid in detection of nucleic acid from SARS-CoV-2, not for any other viruses or pathogens.

This self-sample collection kit is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Federal Food, Drug, and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the authorization is terminated or revoked sooner.

color

COVID-19 Self-Swab Collection Kit

On-site Instructions

color

For In Vitro Diagnostic Use, For Prescription Use Only, For Use Under Emergency Use Authorization Only