This product has not been FDA cleared or approved, but has been authorized for emergency use by FDA under an EUA;

This product has been authorized only for the home collection and maintenance of anterior nasal swab specimens as an aid in detection of nucleic acid from SARS-CoV-2, not for any other viruses or pathogens; and,

The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of medical devices under Section 564(b)(1) of the Federal Food, Drug and Cosmetic Act, 21 U.S.S. §360bbb-3(b)(1), unless the declaration is terminated or authorization is revoked sooner.



Distributed by RapidRona Inc. www.rapidrona.com

If you have questions during your specimen collection, please contact info@myhealthpro.com



## Self-Collection Kit

## INSTRUCTIONS FOR USE

## **PRECAUTION - TIME SENSITIVE TEST - READ BELOW BEFORE STARTING**

FedEx delivery is not available some evenings, Saturdays, Sundays, and national holidays. Please check with your local FedEx for return delivery options before using this kit.

Check FedEx locations: www.fedex.com/en-us/shipping.html

To get valid diagnostic test results, it is important to ship your sample the same day it is taken.

Schedule a pick-up or drop off: www.rapidrona.com/instructions

Follow the sample and packaging instructions closely to prevent inaccurate results or rejected samples.

Collect and package your sample: www.rapidrona.com/instructions

## **TEST KIT CONTENTS**









Tube

label



Pre-filled saline

collection tube







Biohazard Lab sample box

Shipping envelope

Instructions for Use Manual




Rapid Rona LAB SAMPLE BOX TUBE LABEL

Login and register your test kit online: www.rapidrona.com/instructions.

Enter the barcode found on top of the sample box when prompted. Select your return delivery preference: scheduled FedEx pickup or drop off at a local FedEx Dropbox. If you are under the age of 18 please ask your parent or quardian for help with this kit. Children 14-17 can self-collect under adult supervision, while children ages 2-13 require adult assistance.



Place kit contents on a clean, dry surface with enough room to collect your sample.

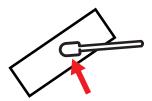


Remove sample swab from its packaging. Do not touch the tip and do not set the swab down.



Wash your hands for at least 20 seconds and drv.



Insert the collection tip of the swab into one nostril until it can't be seen (usually about 1/2 inch to 3/4 inch). If you are collecting a sample from a younger child, insert swab less than 1/2 inch). Rotate swab firmly against the inside wall of the nostril 4 times taking about 15 seconds to collect the sample.

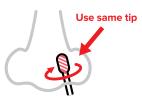


Repeat step 5 in your other nostril, using the same tip.




Wash your hands for at least 20 seconds and dry.



Place tube into biohazard bag with absorbent pad. Seal the bag.

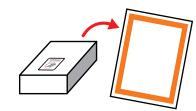


12 Place lab sample box into shipping envelope and seal.

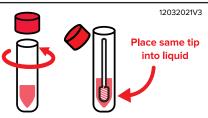

Open pre-filled saline collection tube provided and place sample swab in tube, tip first into the liquid. Screw cap back on.



Using a pen, neatly write your first & last name AND date of birth on tube label. Peel off the sticker back and place the label length-wise on the tube.

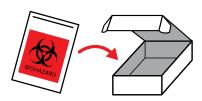

Place biohazard bag into lab sample box and insert tabs to seal box.



Ship your sample SAME DAY OF COLLECTION. Whether you selected a FedEx pick up or Drop Off during registration, be sure to get your sample to FedEx the same day of collection.

\* FAILURE TO SHIP OR DROP OFF THE SAMPLE SAME DAY OF COLLECTION WILL RESULT IN SAMPLE

REJECTION