

# FRONT

color

## Color COVID-19 Self-Swab Collection Kit DTC

1. Activate your kit online.
2. Swab your nose to collect your sample.
3. Package the sample and return your kit.
4. Get results over the phone or computer.

Important: You'll need access to the internet, email, and a smartphone or computer to use this kit.

# BACK

### Your kit includes:

Barcode card  
Swab  
Collection tube  
Biohazard bag  
Return shipping mailer  
Instructions for use  
Fact sheet

### Expiration date:

### Kit storage temperature:

59 to 77°F (15 to 25°C)

For IVD Use, For use under emergency use  
authorization only, Must be 18+ to use this kit



PLASTIC WASTE

This product has not been FDA cleared or approved, but has been authorized for emergency use by FDA under an EUA;

This product has been authorized only for the home collection and maintenance of anterior nasal swab specimens as an aid in detection of nucleic acid from SARS-CoV-2, not for any other viruses or pathogens; and,



The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of medical devices under Section 564(b)(1) of the Federal Food, Drug and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated or authorization is revoked sooner.

DaklaPack<sup>®</sup>  
group

## Before you get started:

- Don't forget to activate your kit.
- Make sure you wash your hands!

## Your kit includes:

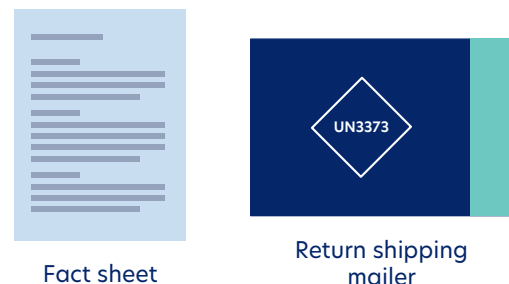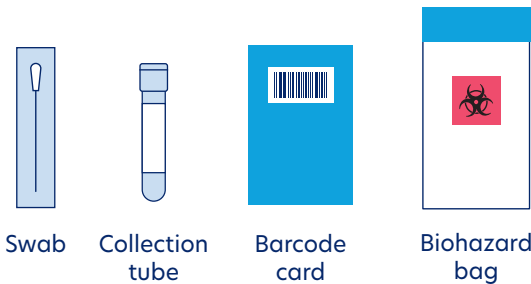

## Checklist before returning your kit:

- Did you activate your kit?
- Did you collect a sample from both nostrils?

Visit [color.com/sample-return](https://color.com/sample-return) to learn more.

This product has not been FDA cleared or approved, but has been authorized for emergency use by FDA under an EUA;

This product has been authorized only for the home collection and maintenance of anterior nasal swab specimens as an aid in detection of nucleic acid from SARS-CoV-2, not for any other viruses or pathogens; and,

The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of medical devices under Section 564(b)(1) of the Federal Food, Drug and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated or authorization is revoked sooner.

## COVID-19 Self-Swab Collection Kit DTC

At-home Instructions

For IVD Use, For use under emergency use authorization only, Must be 18+ to use this kit

1

**Activate your kit online at [color.com/covid/activate](https://color.com/covid/activate).**


You need your unique barcode, which you'll find on your collection tube.



**Warning:** Your sample **must** be dropped off on the same day you activate and collect it.

2


**Before collecting your sample, wash your hands thoroughly for 20 seconds and dry them well.**

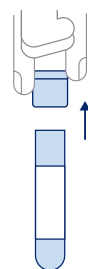

3

**Remove the cap of the collection tube.**

Keep the cap and collection tube nearby.

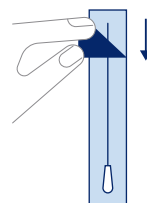


4

**Open the swab package.**

Starting from the handle end of the swab package, peel open the paper backing. Pull swab out of its packaging by the handle.

**Note:** Do not touch the soft tip with your hands or lay it down on any surface.

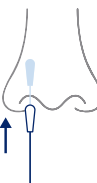

5

**Rotate swab tip in first nostril, 4 times.**

Insert the soft tip into one nostril, until it is no longer visible. Using medium pressure against the inside of your nostril the whole time, rub the swab in a large circular motion at least 4 times.

**Note:** Swab should not go further than 1/2 inch.



6

**Rotate swab tip in other nostril, 4 times.**

Repeat the previous step in the second nostril, using the same end of the swab.


**Note:** You will use the same swab for both nostrils.



7

**Put swab into the collection tube.**

The soft tip of the swab that went into your nose should go into the tube first.

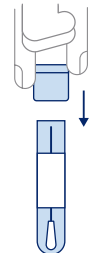

8

**Press the cap of the collection tube firmly closed.**

You're almost done! Make sure the top is closed tightly.

**Note:** After closing the collection tube, wash your hands thoroughly for 20 seconds and dry them well.



9

**Put collection tube into the biohazard bag.**

Close the bag with the adhesive seal.



10

**Put biohazard bag into the return shipping mailer.**

Remove the adhesive strip and seal the mailer closed. This packaging is specially designed for carrying samples suspected of carrying infectious substances. Do NOT replace it.

**Note:** Your return shipping has already been paid.

11

**Return kit.**

Your sample must be returned on the same day you collect it.

**Tip:** If you aren't able to return your sample on the same day, contact [mycovidtest@color.com](mailto:mycovidtest@color.com).

**Questions?**

Visit [support.color.com](https://support.color.com) or contact us at [mycovidtest@color.com](mailto:mycovidtest@color.com) or (844) 352-6567.

## Before you get started:

---

- Your sample must be dropped off on the same day you collect it.
- **Make sure there is a collection today.**  
Refer to your program details if you're not sure where to return your sample.
- **Don't miss the last pickup.**  
Refer to your program details for pickup times.
- **Make sure you wash your hands!**

## Checklist before returning your kit:

---

- Did you activate your kit?
- Did you collect a sample from both nostrils?

Visit [color.com/sample-return](https://color.com/sample-return) to learn more.

---

## Questions?

Visit [support.color.com](https://support.color.com) or contact us at [mycovidtest@color.com](mailto:mycovidtest@color.com) or (844) 352-6567.

This product has not been FDA cleared or approved, but has been authorized for emergency use by FDA under an EUA;

This product has been authorized only for the home collection and maintenance of anterior nasal swab specimens as an aid in detection of nucleic acid from SARS-CoV-2, not for any other viruses or pathogens; and,

The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of medical devices under Section 564(b)(1) of the Federal Food, Drug and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated or authorization is revoked sooner.

The logo for 'color' features the word in a lowercase, sans-serif font. The letter 'o' is replaced by a circular icon composed of several colored segments: yellow, orange, red, purple, blue, and green.

---

## COVID-19 Self-Swab Collection Kit DTC

On-site Instructions

The logo for 'color' features the word in a lowercase, sans-serif font. The letter 'o' is replaced by a circular icon composed of several colored segments: yellow, orange, red, purple, blue, and green.

For IVD Use, For use under emergency use authorization only, Must be 18+ to use this kit

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
## Your kit includes:




Swab



Collection tube



Barcode card

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

Fact sheet



Biohazard bag

1

Activate your kit online at [color.com/covid/activate](https://color.com/covid/activate).

You need your unique barcode, which you'll find on your collection tube.



**Warning:** Your sample will not be tested if the kit is not activated.


2

Before collecting your sample, wash your hands thoroughly for 20 seconds and dry them well.

3

Remove the cap of the collection tube.

Keep the cap and collection tube nearby.

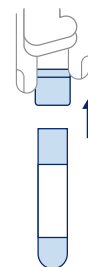


4

Open the swab package.

Starting from the handle end of the swab package, peel open the paper backing. Pull swab out of its packaging by the handle.

**Note:** Do not touch the soft tip with your hands or lay it down on any surface.

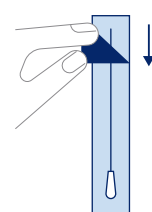


5

Rotate swab tip in first nostril, 4 times.

Insert the soft tip into one nostril, until it is no longer visible. Using medium pressure against the inside of your nostril the whole time, rub the swab in a large circular motion at least 4 times

**Note:** Swab should not go further than ½ inch.

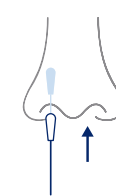

6

Rotate swab tip in other nostril, 4 times.

Repeat the previous step in the second nostril, using the same end of the swab.

**Note:** You will use the same swab for both nostrils.



7

Put swab into the collection tube.

The soft tip of the swab that went into your nose should go into the tube first.




8

Press the cap of the collection tube firmly closed.

You're almost done! Make sure the top is closed tightly.

**Note:** After closing the collection tube, wash your hands thoroughly for 20 seconds and dry them well.

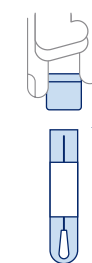

9

Put collection tube into the biohazard bag.

Seal the bag closed.



10

Bring the biohazard bag to your designated test collection point.

Refer to your program details if you're not sure where to return your sample.

**Note:** Don't miss the last pickup time.