



## Before you start

Carefully read all instructions before you begin.

**This kit can be used for individuals 2 years and older.**

For 18 years and older, self-collect sample.

For 14 years and older, self-collect sample under adult supervision.

For 2 years and older, collect sample with adult assistance.


**Sample collection takes less than 10 minutes.**

**The return box must be used to ship sample to the lab.** Check box for pre-applied shipping label. Make sure corners are not damaged and sides have no tears or holes. If damaged, discard and get a new kit by visiting [AmazonDx.com/FAQs](https://www.amazon.com/FAQs)

**Drop off the sample at an eligible UPS location on the same day as collection.** Please collect and drop off Monday - Friday due to UPS shipping schedules. Before registering your kit, confirm the last air drop off time at <http://www.ups.com/dropoff>

Expired samples cannot be processed.

### Components:

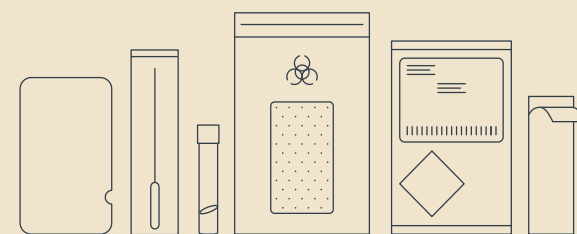

(above left to right) Instructions for use, Nasal swab, Collection tube with saline, Biohazard plastic bag with absorbent pad, Return box with shipping label, Tape

Fact sheet for individuals (not pictured)

# 1

## Register your kit

▶ Watch the overview video during the kit registration process. Rewatch the video at: [AmazonDx.com/FAQs](https://www.amazon.com/FAQs)



Visit AmazonDx.com

Using your personal device, visit AmazonDx.com or view the QR code with your camera app.

Sign in or create an account.



### Enter the collection tube ID

Confirm the test taker's personal information online. Then enter the collection tube ID, found on the side of the collection tube.



### Press "Register kit"

Collect the sample immediately after registering.

⚠ The kit must be registered to get results.

# 2

## Collect your sample

**We recommend testing while seated.**

**If you are collecting the sample for a child, explain the process step-by-step in advance so they know what to expect.**

**Do not touch the soft tip of the swab.**

**Do not touch or drink the fluid in the collection tube.**

**If the fluid touches eyes or skin, rinse with water.**

**If you spill the fluid or the swab touches any surface other than inside the test taker's nostrils, discard the kit components. Clean the area with your routine household surface disinfectant, and get a new kit by visiting [AmazonDx.com/FAQs](https://www.amazon.com/FAQs)**




### Wash and dry hands.

Wash your hands with soap and warm water for 20 seconds.

Use hand sanitizer if there is no sink.


Clean the surface of the area where the sample will be collected.

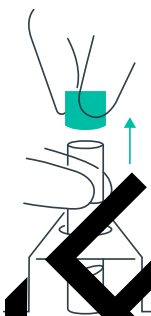

### Remove the lid from the collection tube

Put the tube lid on a clean surface nearby, with the lid top facing down.

Put the collection tube in the tube holder provided in the box to avoid spilling fluid.

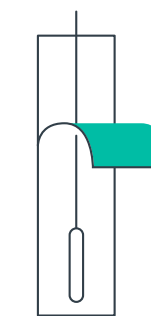


### Open the swab pouch

Peel back the tab on the handle end to remove the nasal swab from the pouch.

Hold by the handle, do not touch the soft tip.

Discard the swab pouch.

⚠ Do not touch the soft tip of the swab to any surface other than inside the test taker's nostrils.

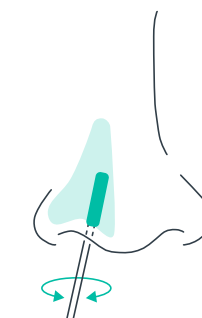

### Swab first nostril


For individuals 14 years or older, insert the soft tip of the swab 1/2 - 3/4 inch inside the nostril.

For children 2-13 years old, insert the swab no more than 1/2 inch.

Rub the swab in a slow circular motion against the inside walls of the nostril at least 4 times.

This should take about 15 seconds.

⚠ If collecting a sample from a child, be careful not to insert the swab too far into the child's nostril. If you feel resistance, stop.

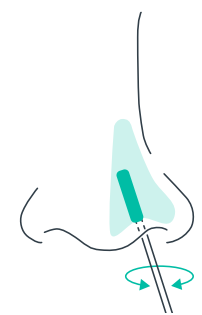

### Repeat in other nostril

In the other nostril using the same swab, rub the swab slowly at least 4 times taking about 15 seconds.




### Put swab in collection tube soft tip down

Put the swab in the tube so the soft tip reaches the bottom.

To break off the swab handle, hold the tube firmly. Bend the swab handle at the indicator line back and forth against the collection tube until it breaks.

Discard the handle.

⚠ Do not spill the fluid in the collection tube! If you spill the fluid, clean the area and discard your kit components.

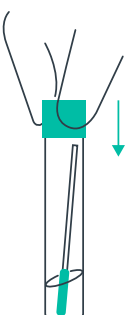

### Recap the collection tube

Make sure the lid is on tight, so no fluid can spill out. If the collection tube leaks your sample cannot be processed.

Wash your hands or use hand sanitizer if there is no sink.

CONTINUE TO NEXT SIDE →

