



## Before you get started:

- Your sample must be dropped off on the same day you activate and collect it.
- **Make sure there is a collection today.** Refer to your program details if you're not sure where to return your sample.
- **Don't miss the last pickup.** Refer to your program details for pickup times.
- Make sure you wash your hands before and after collection.

## Your kit includes:

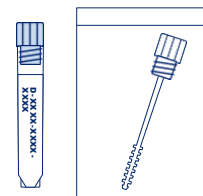

Collection device  
(tube and swab)

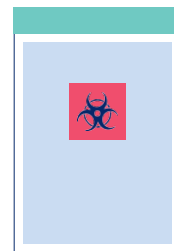

Biohazard bag with  
Absorbent Pad



Barcode  
card

## Checklist before returning your kit:

- Did you activate your kit?
- Did you collect a sample from both nostrils?

Visit [vaulthealth.com/covid/consumer#faq](https://vaulthealth.com/covid/consumer#faq) to learn more.

## Questions?

Visit [vaulthealth.com/covid/consumer#faq](https://vaulthealth.com/covid/consumer#faq) or contact us at [covid@vaulthealth.com](mailto:covid@vaulthealth.com) or (212) 880-5494.

This product (collection kit in combination with the authorized test) has not been FDA cleared or approved, but has been authorized for emergency use by FDA under an EUA for use by the authorized laboratory.

This product has been authorized only for the detection of nucleic acid from SARS-CoV-2, not for any other viruses or pathogens; and,

The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Federal Food, Drug and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated or authorization is revoked sooner.

## RhinoStics Nasal Swab Collection Kit

### for Nexus High Throughput SARS-CoV-2 Assay

#### On-site Instructions

For In Vitro Diagnostic Use, For Prescription Use Only, For Use Under Emergency Use Authorization Only  
For use by individuals 18 years and older.

1

**Don't forget:**  
Activate your kit online  
at [pcr.vlt.co](http://pcr.vlt.co).

You need your unique barcode,  
which you'll find on your collection  
tube.



**Warning:** Your sample will not be  
tested if the kit is not activated.

2


Before collecting your sample,  
wash your hands thoroughly for  
20 seconds and dry them well.

3




Unscrew the cap from the  
collection tube.  
You can discard the cap.

4

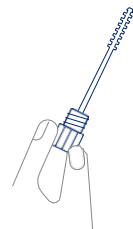

Remove the swab from  
its bag.  
Do not touch the ridged tip  
with your hands or lay it  
down on any surface.


5

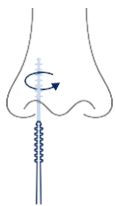

Rotate swab tip in  
first nostril, 3 times.  
Insert the swab into one nostril, until you  
feel resistance. Using medium pressure  
against the inside of your nostril the whole  
time, rub the swab in a large circular  
motion at least 3 times.


**Note:** Swab should not go further than 1 inch.

6

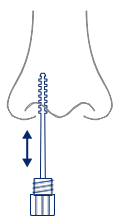


Gently slide the swab up and  
down against the inside of your  
nostril 3 times.  
Keep medium pressure on the swab against  
the inside of your nostril.

7

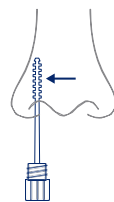

Hold the swab against  
the inside of your nostril.  
Keep it there for about  
10 seconds.


8

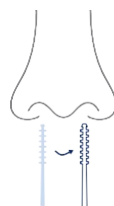

Repeat steps 5 to 7 in the  
other nostril.  
Rotate at least 3 times, slide the  
swab up and down, then hold  
the swab against your nostril for  
10 seconds.

**Note:** You will use the same end  
of the swab for both nostrils.

9

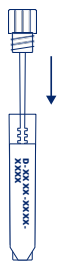

Put swab into the  
tube portion of the  
collection device.  
The tip of the swab that went  
into your nose should go into  
the tube first.

10



Screw the cap closed tightly.  
Make sure the top is carefully  
screwed closed.  
**Note:** After screwing the collection  
device closed, wash your hands  
thoroughly for 20 seconds and dry  
them well.

11



Put collection device  
into the biohazard bag.  
Seal the bag closed.



12

Bring the biohazard bag to your  
designated test collection point  
on the day you collected your  
sample.  
Refer to your program details if you're  
not sure where to return your sample.  
**Note:** Don't miss the last pickup time.

## Before you get started:

- Your sample must be dropped off on the same day you activate and collect it.
- Refer to your program details if you're not sure where to return your sample.
- Don't miss the last pickup. Refer to your program details for pickup times.
- Make sure you wash your hands before and after collection.

## Your kit includes:

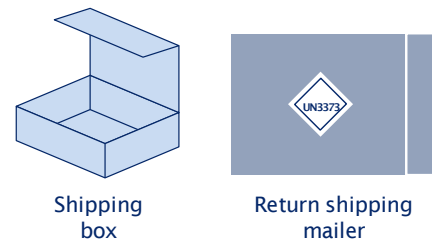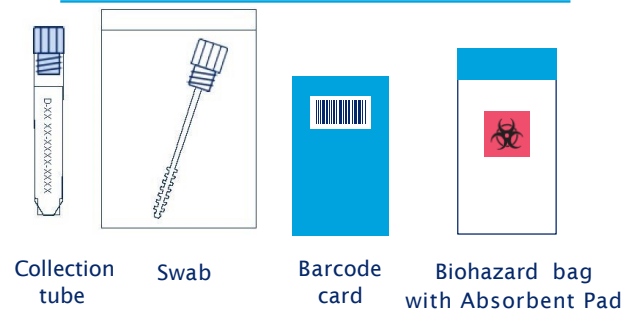

## Checklist before returning your kit:

- Did you activate your kit?
- Did you collect a sample from both nostrils?

Visit [vaulthealth.com/covid/consumer#faq](https://vaulthealth.com/covid/consumer#faq) to learn more.

## Questions?

Visit [vaulthealth.com/covid/consumer#faq](https://vaulthealth.com/covid/consumer#faq) or contact us at [covid@vaulthealth.com](mailto:covid@vaulthealth.com) or (212) 880-5494.

This product (collection kit in combination with the authorized test) has not been FDA cleared or approved, but has been authorized for emergency use by FDA under an EUA for use by the authorized laboratory.

This product has been authorized only for the detection of nucleic acid from SARS-CoV-2, not for any other viruses or pathogens; and,

The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Federal Food, Drug and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated or authorization is revoked sooner.

## Rhinostics Nasal Swab Collection Kit

### for Nexus High Throughput SARS-CoV-2 Assay

Mail-In Instructions

For In Vitro Diagnostic Use, For Prescription Use Only, For Use Under Emergency Use Authorization Only  
For use by individuals 18 years and older.

Version 1.0

1

**Don't forget:**  
Activate your kit online at [pcr.vlt.co](http://pcr.vlt.co).

You need your unique barcode, which you'll find on your collection tube.



**Warning:** Your sample will not be tested if the kit is not activated.

2

Before collecting your sample, wash your hands thoroughly for 20 seconds and dry them well.



3

Unscrew the cap of the collection tube.

You can discard the cap.

4

Remove the swab from its bag.

Do not touch the soft tip with your hands or lay it down on any surface.




5

Rotate swab tip in first nostril, 3 times.

Insert the swab into one nostril, until you feel resistance. Using medium pressure against the inside of your nostril the whole time, rub the swab in a large circular motion at least 3 times.

**Note:** Swab should not go further than ½ inch.

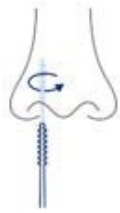

6

Gently slide the swab up and down against the inside of your nostril 3 times.

Keep medium pressure on the swab against the inside of your nostril.



7

Hold the swab against the inside of your nostril.

Keep it there for about 10 seconds.




8

Repeat steps 5 to 7 in the other nostril.

Rotate at least 3 times, slide the swab up and down, then hold the swab against your nostril for 10 seconds.

**Note:** You will use the same end of the swab for both nostrils.

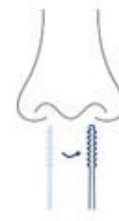

9

Put swab into the tube portion of the collection device and screw cap closed tightly.

The tip of the swab that went into your nose should go into the tube first. **Make sure the cap is closed tightly.**

**Note:** After screwing the collection device closed, wash your hands thoroughly for 20 seconds and dry them well.



10

Put the collection tube into the biohazard bag.

Seal the biohazard bag closed.

**Note:** The collection tube should go within the rigid plastic insert in the biohazard bag.




11

Put biohazard bag into the shipping box.

Seal the shipping box closed.

**Note:** Use the same box that your kit came in.

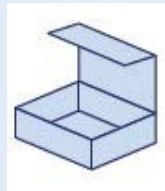

12

Put box into the return shipping mailer.

Seal the return shipping mailer closed. Your return shipping has already been paid.

**Note:** This packaging is specially designed for shipping samples suspected of carrying infectious substances. Do NOT replace it.



13

Return kit.

Your sample must be returned on the same day you collect it.

**Note:** If you aren't able to return your sample on the same day, contact [covid@vaulthealth.com](mailto:covid@vaulthealth.com).