

Useful Handling Tips for Ecoflac® plus

Infusion Administration



Opening

Peel off the covering foil of port.



Injection Port

Swab injection port.



Handling during spiking

When inserting an infusion set or a needle into the port, grip the neck of the Ecoflac plus container to stabilize the container or leave it free standing.




Using infusion sets

Always insert the infusion set at a right (90°) angle. When the set is removed, the port on Ecoflac plus reseals automatically. **Avoid rotating the spike when piercing the port.**



No venting required

During infusion, Ecoflac plus container collapses completely without any need for ventilation.

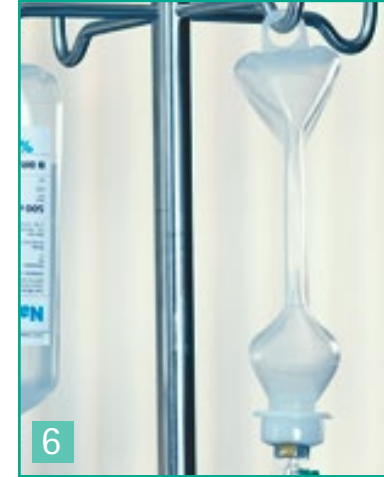

Collapsed Ecoflac plus